# PULSE CLEANER

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YOU CAN OBTAIN A **PROPER CLEANING** OF THE ASPIRATION SYSTEM WITH:

- **"Pulse Cleaner"**, the container creating the necessary turbulence,
- Puli-Jet plus 2.0 detergent and disinfecting **with antiscalant agent**,
- antifoaming tablets, preventing the foam build-up during the dentist's operations

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**"Pulse Cleaner" is a brand-new turbulence-maker. The pulsating device creates a turbulence which removes dirt and bacterial patina from the piping. The staff in charge of the aspiration system cleaning wastes no time:** after having prepared the solution of Puli-Jet plus 2.0 at 0.8% with warm water (about 50 °C) to clean and disinfect, place the terminals on the pistons. Cleaning aspiration will start without any further operation. When the solution has been almost used up, the aspiration noise changes thus indicating that the cleaning procedure has finished; terminals can be replaced on their seats. The solution left on the bottom of the container should be used to disinfect the spittoon. Do not rinse the aspirator, do not turn on the spittoon-rinsing tap: Puli-Jet plus 2.0 will go on dissolving blood and mucus and disinfecting the system all through the night.

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Company with Quality System certified = ISO 9001 = and = ISO 13485 =

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**HOW IS IT WE LEAD IN OUR FIELD, WHEN WE COST LESS THAN THE ALTERNATIVES? THIS IS HOW:**

**Constant research:** this enables us to apply the latest technology to all of our products and solutions.

**We enhance performance:** electronic and information technology enable us to enhance the performance and reliability of our products.

**We reduce costs:** less maintenance and lower energy costs mean that we are always the most economical on a cost-benefit analysis.

**We reduce environmental impact:** we save 50% on raw materials, and allow you to save between 30% and 50% on electrical consumption.

Via Natta 6/A 43122 Parma – Italy T +39 0521 607604 info@cattani.it

**WWW.CATTANI.IT**