**Lavoro** BOOKLET MULTIPAGINA ECO  
JET 1 TISSUE

**Data** 27-10-2017

**Data approv.**  Vedi doc. allegato

**Bozza n°** 2

**Dimensioni** 120x80 / 90x125 mm

78,5

FRONTE

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**Danger.** Flammable liquid and vapor. Harmful to aquatic life with long lasting effects. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN (or hair): Take off immediately with soap and water. Dispose of contents/container in accordance with local/regional/national/international regulations.

all contaminated clothing. Rinse skin with water / shower. Dispose of contents/container in accordance with local/regional/national/international regulations.

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2 "ECO-JET 1 tissue" is a high-level disinfectant recommended for cleaning and regular disinfection of small-size surfaces and objects including medical devices (such as armchair armrests, surgical light handles, drawers handles, handpieces, shelves, trays etc...)

**INSTRUCTION FOR USE:** Remove the dispenser cover, break covering seal, hold the cover of the wipe showing from the wipes roll and thread through the star-shaped opening of the lid corner. Replace the lid cover. The wipes can be extracted automatically, snap each one by pulling at an angle.

**EFFICACY/KILL TIME:** After 15 minutes contact they are certified according to official analytical methodology: bactericidal and fungicidal activity [EN 13697], mycobactericidal/tubercutocid activity [EN 14348] and virucidal activity (HIV, HBV and HCV) [EN 14476].

**COMPOSITION:** 100 g active wetting solution contain about: 3-butoxy-2-propanol [CAS 5131-66-8] 0.6%, Didecyl dimethyl ammonium chloride [CAS 7173-51-5] 0.56%, propan-2-ol 0.22% [CAS 67-63-0], isopropyl alcohol [CAS 67-63-0] <10%, perfume and excipients q.s. for 100 g

**WARNINGS:** After opening, always reseal the cap of the star-shaped throat to prevent the product from losing moisture. Do not use the wipes if they are not sufficiently wet or if dried out. In case of dropping or breaking the container with subsequent possible drying out of the wipes, the product must not be used. Check the container's integrity before use. Contain possible small drips of product using the specific PPE (gloves). Generally, the product is compatible with all materials, including the chair's upholstery, thanks to its low alcohol content; however, a preventive test on a small part of the surface to be treated, not in prominent position, is recommended. Do not mix or add other products to the container. Do not use the product with other mixtures or solutions. The sealed product must be stored in a fresh and dry place and in all cases at a temperature lower than 50°C. Use the product within 3 months from opening and before the expiration date shown on the dispenser. Always check the integrity of the container before use. The expiration date refers to the correct storage of the product in a sealed container. Use the proper PPE to handle the product (gloves, glasses, mask, etc.). Before using read carefully the safety sheet and the product specifications downloaded from the web site: [www.cattani.it](http://www.cattani.it).

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