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DANGER. It causes severe burns on the skin and severe lesions to the eyes. May cause an allergic skin reaction. Toxic for aquatic organisms with long lasting effects. Do not breathe the vapours. Avoid release to the environment. Wear protective gloves and clothes and protect eyes and face. IN CASE OF CONTACT WITH THE SKIN (or hair): remove immediately all the contaminated clothes. Rinse the skin (or shower). IN CASE OF CONTACT WITH THE EYES: rinse carefully for some minutes. Remove contact lenses, if easy to do. Continue rinsing. Contact immediately a POISON CENTRE or a physician. Contains: Sodium hydroxide, Chlorocresol, Tetrasodium EDTA, Polyoxy-1,2-ethanediol-phenyl-hydroxy phosphate, Citral.

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Puli-Jet Plus 2.0 is a non-foaming, aldehyde and phenol-free concentrate for disinfection, deodorization, cleaning and maintenance of all types of dental suction units (liquid ring, wet ring and air). The system should be disinfected every night, at the end of the working day and a cleaning wash should be performed at least once half through the day. It is recommended also to perform a disinfection of the equipment after working on patients at high infection risk. It is always good practice to refer to the evaluation of a professional before implementing cleaning and disinfection cycles, so as to minimize the risk of contamination. To prevent stains and corrosion, wipe off any spills from the bottle before replacing it after use. In case of accidental spilling of the pure or diluted product, wipe the surface dry immediately and clean with a detergent that is compatible with that surface in order to prevent stains and corrosion. **DIRECTIONS FOR USE:** 1) Filling the dispenser: keep the bottle vertical, preferably resting it on a surface, loosen the red cap, press the bottle gently on the two labels and fill the dispenser up to the 20 cm³ mark, to be mixed with 2,5 L of hot water (40-50°C) which is the content of the Pulse Cleaner when filled up to the highest position of the siphons. For ordinary cleaning: just mix 10 cm³ of Puli Jet Plus 2.0 with 2,5 L of water. Once a week, possibly before the weekend or for longer breaks, a cleaning cycle with a double dose of 40 cm³. 2) Blend the solution to homogenize, insert the terminals in the Pulse Cleaner using the special adapters, if needed, and draw the solution. **Contains:** 10-16% Chlorocresol, 10-12% 1-Methoxy-2-Propanol, 3,17-5,17% Sodium hydroxide, 2,74-4,74% Tetrasodium EDTA, 1,88-3,88% propan-2-ol, 1,5-3,5% Polyoxy-1,2-ethanediol-phenyl-hydroxy phosphate, 0,11-0,31% Citral, 0,05-0,15% Phosphoric acid. **Spectrum of activity:** at 0,8% concentration, Puli-Jet Plus 2.0 proved to have bactericidal [EN 13727] and fungicidal [EN 13624] effectiveness. **Warnings:** Extremely low temperatures can cause the product to freeze. Even in the event of freezing, the effectiveness of the product will not be affected. Return to liquid state by placing it at 25 - 30 °C room temperature, making sure to shake the vial before use in order to homogenize the product. Mix the solution prepared just before use for a proper solubilisation of the product in the solution. Browning of the liquid is natural and will not affect its detergent and disinfectant effectiveness. Should the bottle fall, break and the product be spilled, use appropriate PPE (gloves, clothing, protective glasses, etc.) and promptly wipe up the product. Then clean with a detergent to prevent stains or corrosion of surfaces. The solution should be prepared in a clean container in a suitable room with no risk of contamination. Shake the bottle before use in case of component stratification. Store in a dry, cool place. Use the product within 3 months from opening and before the expiry date shown on the label. After opening, keep the bottle closed between uses. Always check the integrity of the container before use. Do not dilute and/or mix with other products. The expiry date refers to correctly stored sealed packages. Use appropriate PPE to handle the product (gloves, glasses, mask, etc.). Before use, carefully read the product's safety sheet and specifications download from the website: www.cattani.it

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